LifeStyles condoms are available:

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Non-lubricated d the preferred dry condom

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Lubricated with Spermicide* (Nonoxonyl-9) for extra protection

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Condoms lubricated with Nonoxonyl-9 combine a latex condom and a spermicidal lubricant. The spermicide, Nonoxonyl-9, reduces the number of active sperm, thereby decreasing the risk of pregnancy if you lose your erection before withdrawal and some semen spills outside the condom. However, the extent of decreased risk has not been established. This condom should not be used as a substitute for the combined use of a vaginal spermicide and a condom.

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SXWELL USA LLC,
111 Wood Ave South
Iselin, NJ 08830

www.lifestyles.com

© and ™ are trademarks owned by LifeStyles Healthcare Pte Ltd. ©2018 LifeStyles Healthcare Pte Ltd
Patents: www.lifestyles.com/our-patents

NPKL00001
169703

© 2018 LifeStyles Healthcare Pte Ltd

SUMMARY TABLE OF CONTRACEPTIVE EFFICACY

Percentage of women experiencing an unintended pregnancy during the first year of typical use and the first year of perfect use of contraception and the percentage continuing use at the end of the first year. United States.

Method (1)	% of Women Experiencing an Unintended Pregnancy within the First Year of Use		% of Women Continuing use at One Year
	Typical Use 1 (2)	Perfect Use 2 (3)	
No method 4	85	85	
Spermicides 5	29	18	42
Withdrawal	27	4	4 3
Fertility Awareness based methods	25	5	1
Standard Days Method 6	5		
TwoDay method 8	4		
Ovulation method 6	3		
Sponge			
Parous women	32	20	46
Nulliparous women	1 6	9	57
Diaphragm 7	16	6	5 7
Condom 8			
Female (Reality)	2 1	5	49
Male	15	2	5 3
Combined pill and progestin-only pill	8	0 3	68
Evra Patch	8	0 3	68
NuvaRing	8	0 3	68
Depo-Provera	3	0 3	56
IUD			
ParaSard (copper T)	0 8	0 6	78
Mirena (LNG-IUS)	0 2	0 2	80
Implanon	0 05	0 05	84
Female Sterilization	0 5	0 5	100
Male Sterilization	0 15	0 10	100

Emergency Contraceptive Pills: Treatment initiated within 72 hours after unprotected intercourse reduces the risk of pregnancy by at least 75%. 9
Lactational Amenorrhea Method: LAM is a highly effective, temporary method of contraception. 10

Source: Trussell J. Contraceptive efficacy. In Hatcher RA, Trussell J, Nelson AL, Cates W, Stewart FH, Kowal D. Contraceptive Technology. Nineteenth Revised Edition. New York NY: Ardent Media, 2007.

1 Among typical couples who initiate use of a method (not necessarily for the first time), the percentage who experience an accidental

pregnancy during the first year if they do not stop use for any other reason. Estimates of the probability of pregnancy during the first year of typical use for spermicides, withdrawal, periodic abstinence, the diaphragm, the male condom, the pill, and Depo-Provera are taken from the 1995 National Survey of Family Growth corrected for underreporting of abortion; see the text for the derivation of estimates for the other methods.

2 Among couples who initiate use of a method (not necessarily for the first time) and who use it perfectly (both consistently and correctly), the percentage who experience an accidental pregnancy during the first year if they do not stop use for any other reason. See the text for the derivation of the estimate for each method.

3 Among couples attempting to avoid pregnancy, the percentage who continue to use a method for 1 year.

4 The percentages becoming pregnant in columns (2) and (3) are based on data from populations where contraception is not used and from women who cease using contraception in order to become pregnant. Among such populations, about 89% become pregnant within 1 year. This estimate was lowered slightly (to 85%) to represent the percentage who would become pregnant within 1 year among women now relying on reversible methods of contraception if they abandoned contraception altogether.

5 Foams, creams, gels, vaginal suppositories, and vaginal film.

6 The Ovulation and TwoDay methods are based on evaluation of cervical mucus. The Standard Days method avoids intercourse on cycle days 8 through 19.

7 With spermicidal cream or jelly.

8 Without spermicides.

9 The treatment schedule is one dose within 120 hours after unprotected intercourse, and a second dose 12 hours after the first dose. Both doses of Plan B can be taken at the same time. Plan B (1 dose is 1 white pill) is the only dedicated product specifically marketed for emergency contraception. The Food and Drug Administration has in addition declared the following 22 brands of oral contraceptives to be safe and effective for emergency contraception: Ogestrel or Ovral (1 dose is 2 white pills), Leven or Nordette (1 dose is 4 light-orange pills), Crystelle, Levora, Low-Ogestrel, Lo/Ovral or Quessence (1 dose is 4 white pills), Tri-Leven or Triphasil (1 dose is 4 yellow pills), Jolesse, Portia, Seasonale or Trivora (1 dose is 4 pink pills), Seasonique (1 dose is 4 light-blue-green pills), Empresse (1 dose is 4 orange pills), Alesse, Lessina or Levite (1 dose is 5 pink pills), Aviane (1 dose is 5 orange pills), and Lutera (1 dose is 5 white pills).

10 However, to maintain effective protection against pregnancy, another method of contraception must be used as soon as menstruation resumes, the frequency or duration of breastfeeds is reduced, bottle feeds are introduced, or the baby reaches 6 months of age.

How to use a Condom

In English/Spanish

If used properly, latex condoms are effective against pregnancy, AIDS, (HIV Infection), and other sexually transmitted diseases.

CAUTION: This product contains natural rubber latex which may cause allergic reactions.

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Latex condoms are intended to prevent pregnancy, HIV/AIDS, and other sexually transmitted infections.

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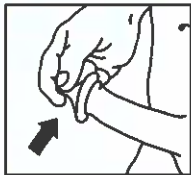
Tear open the package carefully. Do not use fingernails, teeth, or anything that can damage the condom.

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Squeeze tip of condom to remove air. (Excess air could cause condom to break.)

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Before any sexual contact, place the condom on the head of the erect penis with the rolled side out. Lesions, pre-ejaculate secretions, semen, vaginal secretions and blood can all transmit infectious organisms. As sperm can be released prior to ejaculation, the condom should be placed on the penis before any contact of the penis with the vagina.

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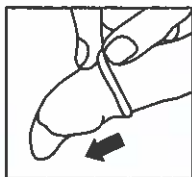
Unroll or pull the condom all the way to the base of the erect penis. If the condom doesn't unroll easily, it may be on backwards, damaged or too old. Throw it away and start over with a new condom.

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Immediately after ejaculation, hold the rim of the condom in place and withdraw the penis while it is still erect. Avoid spilling semen.

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Dispose of a used condom by wrapping it in tissue and throwing it into the trash. Wash your hands with soap and water.

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Avoid further sexual contact with your partner until both of you wash your sex organs and any other areas that came in contact with body fluids.

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Store at room temperature. Protect latex condoms against prolonged periods of exposure to extreme low or high temperatures, moisture, direct sunlight and fluorescent light.

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Store latex condoms in a cool, dry place (below 100°F) and avoid exposure to direct sunlight.

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If the rubber material is sticky or brittle or obviously damaged, do not use the condom.

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If the color is uneven or changed, do not use the condom. Si el color no es uniforme o está cambiado, no use el condón. Make sure there is adequate lubrication. If you add lubricant, use a water-based LifeStyles® lubricant. DO NOT USE OIL-BASED LUBRICANTS, such as those made with petroleum jelly (e.g., Vaseline®), mineral oil, vegetable oil, or cold cream, as these may damage the condom.

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If you or your partner sense burning or itching, discontinue use of this condom. Try another type of condom.

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If the condom breaks or semen (cum) spills or leaks out, you and your partner should wash wherever you had sexual contact right away.

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Latex condoms reduce the risk of transmitting STIs by providing a barrier against the source of the infection.

Latex condoms are most effective against STIs such as HIV infection (AIDS) and gonorrhea that are spread by contact with the head of the penis.

Latex condoms are less effective against STIs such as Human Papillomavirus (HPV) and herpes. These STIs can also be spread by contact with infected skin that is not covered by the condom.

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If you believe you have an STI, contact a health care provider. For more information on latex condoms or STIs, contact a health care provider or a public health agency.

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Although no prophylactic or contraceptive can guarantee 100% effectiveness, latex condoms, if properly used, will help to reduce the risk of transmission of HIV infection (AIDS) and many other sexually transmitted diseases, including chlamydia, genital herpes, genital warts, gonorrhea, hepatitis B, and syphilis. Latex condoms can also prevent pregnancy without the serious side effects sometimes associated with some other methods. **Caution: This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.** Condoms are primarily intended for use in vaginal sex; other uses can increase the potential for breakage.

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4 ADDITIONAL INFORMATION INFORMACION ADICIONAL

Although no contraceptive can guarantee 100% effectiveness, latex condoms, when properly used, are a highly effective means of preventing pregnancy without the serious side effects sometimes associated with some other methods.

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